

Find the ED  
treatment that's  
right for you



AMS 700™  
Inflatable Penile Implant

**EDCure**

Erectile dysfunction can limit your intimacy, affect your self-esteem and impact your relationship with your partner.<sup>1</sup> But there's good news—ED is treatable.

### What is erectile dysfunction (ED)?


ED is defined as the ongoing inability to achieve or maintain an erection that is firm enough to perform sexual intercourse.<sup>2</sup> ED is a common problem, and it's important to know you're not alone.

### What causes ED?<sup>1-3</sup>

ED happens when blood flow to the penis is limited or nerves are damaged. The cause can be physical, psychological, or a combination of both. Psychological causes include stress, anxiety, guilt, and depression. Physical causes include diabetes, vascular disease, side effects of medications and prostate cancer treatment.



### Common conditions alongside ED<sup>5</sup>

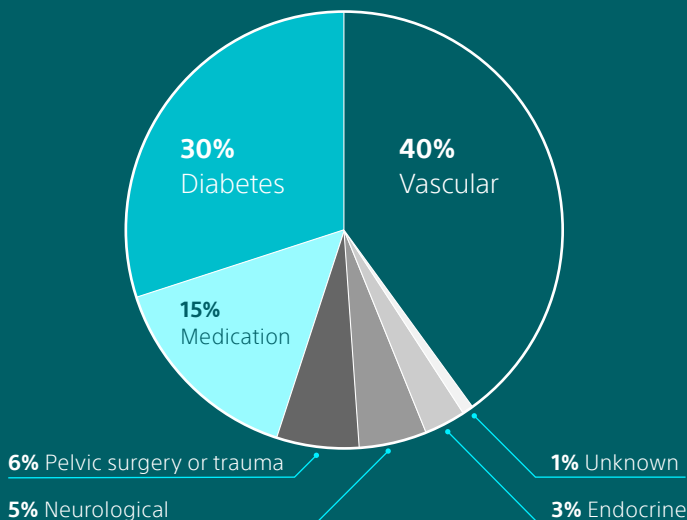

**ED is not just a medical issue;  
it can also deeply affect relationships.<sup>1</sup>**

*“The intimacy that we used to have went away. All of a sudden, it was like we were completely separated. There was no connection.”*

— Tom

Results from case studies are not necessarily predictive of results in other cases. Results in other cases may vary.

## Potential treatment options<sup>2</sup>


**Whatever is causing your ED, there are treatment options. Oral medications are a common first step, but they don't work for everyone or may stop working.**

Other treatments include vacuum erection devices, injections and intraurethral suppositories. A penile implant is designed to be a long-term treatment for men with ED and can provide an erection whenever and wherever desired.

### AMS 700™ Penile Implant

An inflatable penile implant, like the AMS 700 Penile Implant, may be a suitable long-term treatment option for men with erectile dysfunction. A penile implant is a device that is implanted into the penis, and the implant surgery usually doesn't require an overnight hospital stay.

The implant is hidden inside the body, so no one can see it. The user can control it to get and keep an erection for as long desired.

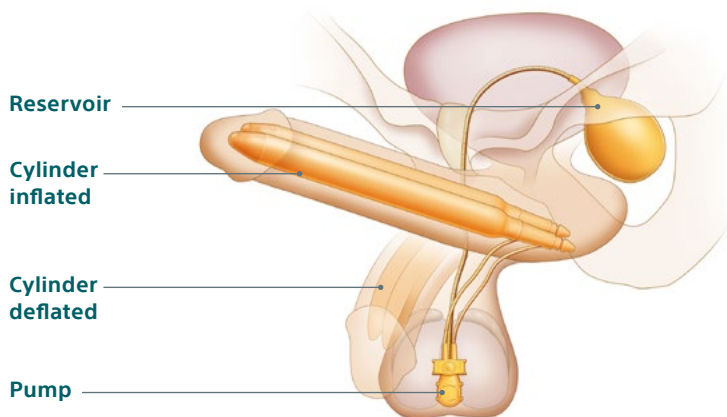

**Penile implants have been in clinical use for over 50 years with over 500,000 Boston Scientific penile implants sold.<sup>7,8</sup>**

## Boston Scientific penile implants

Please talk to your doctor to decide which implant may be right for you.

### AMS 700™

Inflatable Penile Prosthesis with TENACIO™ Pump  
LGX, CX, CXR

*Available 2024 (US)*



### AMS 700™

Inflatable Penile Prosthesis with MS Pump™  
LGX, CX, CXR

*Available 2006 (US)*



### Tactra™

Malleable Penile Prosthesis



Prior to your procedure, your doctor should walk you through the penile implant and how it works. There is a Patient Guide available. It includes the name of the device and pump, as well as step-by-step instructions on how to use it. You can always reach out to your doctor for more information.

## Could a penile implant be right for you?

The AMS 700 Penile Implant is designed to closely mimic a natural erection, providing rigidity when inflated and a natural, flaccid appearance when deflated.

- Designed to be a long-term treatment for ED
- Spontaneous – you can activate the device when the mood strikes
- Once activated, the erection can last as long as desired
- Entirely contained inside the body – no one knows you have an implant unless you tell them
- Designed to feel natural during intercourse and typically does not interfere with ejaculation or orgasm<sup>9-12</sup>
- Broad range of lengths available depending on your anatomy
- Cylinders that expand in length and girth
- Available pre-treated with antibiotics designed to reduce the risk of infection

**Important Note:** Devices pre-treated with antibiotics are not suitable for patients who are allergic to rifampin, minocycline HCl or other tetracyclines, or for patients with systemic Lupus. Talk to your doctor about device options that do not contain antibiotic treatment.

## Risks of a penile implant:

- Organic erections are no longer possible
- Infection, in which case the implant may have to be removed. Certain conditions, such as diabetes, spinal cord injuries or skin infections, may increase infection risk.
- Pain, which is typically associated with the healing process
- Mechanical failure of the implant potentially leading to additional surgery
- Device replacement, lifetime of implant is 10 years
- Unintended inflation of the device

**In one survey of which 194 patients received an AMS 700 Penile Implant with a Momentary Squeeze (MS) Pump:<sup>4</sup>**

**95%** of patients were very satisfied or satisfied with sexual intercourse

**95%** of patients were very satisfied or satisfied with their sexual activity

**In a survey of 80 patients who received an AMS 700 Penile Implant:<sup>6</sup>**

**97%** would recommend a penile implant to a friend

## EDCure

Talk to someone who's been there:

Call us at **1-844-4ED-CURE** or email

**MHPatientEducation@bsci.com** to be introduced to a patient who found a treatment for ED.

**Visit EDCure.com to:**

- Take the online ED quiz, and get your results.
- Find an ED doctor in your area.
- Register for free patient education seminars to learn about treatment options.
- Hear how real people like you have treated ED.
- Get answers to common questions about ED, penile implants and insurance coverage.



## References

1. DiMeo PJ. Psychosocial and relationship issues in men with erectile dysfunction. *Urol Nurs*. 2006;26:442-446.
2. Erectile Dysfunction. National Institute of Diabetes and Digestive and Kidney Diseases. <http://www.nlm.nih.gov/medlineplus/erectiledysfunction.html>. Accessed July 16, 2024.
3. Feldman HA, Goldstein I, Hatzichristou DG, Krane RJ, McKinlay JB. Impotence and its medical and psychosocial results of the Massachusetts Male Aging Study. *J Urol*. 1994;151:54-61.
4. Otero JR, Cruz CR, Gómez BG, et al. Comparison of the patient and partner satisfaction with 700CX and Titan penile prostheses. *Asian J Androl*. 2017 May-Jun;19(3):321-5.
5. Shabsigh R, Lue TF. A Clinician's Guide to ED Management. New York: Haymarket Media Inc.; 2006.
6. Bettocchi C, Palumbo F, Spilotros M, et al. Patient and partner satisfaction after AMS inflatable penile prosthesis implant. *J Sex Med*. 2008;5:1503-1512.
7. Data on file with Boston Scientific.
8. Scott FB, Bradley WE, Timm GW. Management of erectile impotence: use of implantable inflatable prosthesis. *Urology*. 1973;2:80-82.
9. Montorsi F, Rigatti P, Carmignani G, et al. AMS three-piece inflatable implants for erectile dysfunction: a long-term multi-institutional study in 200 consecutive patients. *Eur Urol*. 2000 Jan;37(1):50-5.
10. Levine LA, Becher EF, Bella AJ, et al. Penile Prosthesis Surgery: Current Recommendations From the International Consultation on Sexual Medicine [published correction appears in *J Sex Med*. 2016 Jul;13(7):1145]. *J Sex Med*. 2016;13(4):489-518. doi:10.1016/j.jsxm.2016.01.017
11. Eli Coleman PhD, Alan Listiak PhD, Gordon Braatz PhD & Paul Lange MD (1985): Effects of Penile Implant Surgery on Ejaculation and Orgasm, *Journal of Sex & Marital Therapy*, 11:3,199-205
12. Bae JH, Song PH, Kim HT, Moon KH. Assessment of erectile and ejaculatory function after penile prosthesis implantation. *Korean J Urol*. 2010 Mar;51(3):202-7.

**Caution:** U.S. Federal law restricts this device to sale by or on the order of a physician.

Your doctor is your best source for information on the risks and benefits of the AMS 700™ Inflatable Penile Prosthesis. Talk to your doctor for a complete listing of risks, warnings and important safety information.

The AMS 700™ Inflatable Penile Prosthesis is intended for use in the treatment of male erectile dysfunction (impotence). Implanting a penile prosthesis will damage or destroy any remaining ability to have a natural erection, as well as make other treatment options (oral medications, vacuum devices or injections) impossible.

Men with diabetes, spinal cord injuries or skin infections may have an increased risk of infection. Implantation may result in penile curvature or scarring. Some AMS 700 devices contain an antibiotic (InhibiZone™ Antibiotic Surface Treatment). The device is not suitable for patients who are allergic to the antibiotics contained within the device (rifampin, minocycline HCl or other tetracyclines) or those who have systemic lupus, these patients should use one of the devices that do not contain InhibiZone antibiotic surface treatment.

Potential risks may include: device malfunction/failure leading to additional surgery, device migration potentially leading to exposure through the tissue, wearing away/loss of tissue (device/tissue erosion) infection, unintended-inflation of the device and pain/soreness. MH-545411-AD

All images are the property of Boston Scientific. All trademarks are the property of their respective owners.

## Patient testimonials

The stories throughout this brochure recount the experiences of people who are using Boston Scientific therapies related to erectile dysfunction. Boston Scientific invited these people to share their stories candidly. As you review them, please bear in mind that the experiences are specific to these particular people. As with all medical treatment, not every response is the same. We recommend you talk to your doctor about what treatment is right for you.

The patients featured in this brochure may have been compensated for their travel and/or time. Results from case studies are not necessarily predictive of results in other cases. Results in other cases may vary.

# Boston Scientific

Advancing science for life™

Boston Scientific Corporation  
300 Boston Scientific Way  
Marlborough, MA 01752-1234  
[www.BostonScientific.com](http://www.BostonScientific.com)

©2024 Boston Scientific Corporation  
or its affiliates. All rights reserved.

MH-546505-AB AUG 2024

